HEARTSTART TRAINING PADS II
 ÉLECTRODES DE FORMATION HEARTSTART II
 HEARTSTART SCHULUNGSELEKTRODEN II
 TRAININGSELEKTRODE HEARTSTART II
 ELECTRODOS DE ENTRENAMIENTO HEARTSTART II
 ELETTRODI PER ADDESTRAMENTO HEARTSTART II
 ELETRODOS DE TREINAMENTO HEARTSTART II
 HEARTSTART-ÖVNINGSELEKTRODER II
 HEARTSTART ØVELSESELEKTRODER II
 HEARTSTART-HARJOITTELU-ELEKTRODIT II
 HEARTSTART-TRÆNINGSELEKTRODER II
 ELEKTRODY SZKOLENIOWE HEARTSTART II
 UČEBNÍ NALEPOVACÍ ELEKTRODY HEARTSTART II
 УЧЕБНЫЕ ЭЛЕКТРОДЫ HEARTSTART II
 ΕΚΠΑΙΔΕΥΤΙΚΑ PADS II HEARTSTART
 HEARTSTART EĞİTİM PEDLERİ II
 HEARTSTART 訓練用電極片 II
 HEARTSTART 培训用电极 II
 ハートスタート・トレーニング・パッド II
 HEARTSTART 실습용 패드 II
 แผ่นแพดสำหรับการฝึกหัด ฮาร์ทสตาร์ท II
 HEARTSTART II لاصفا ت التدريب

989803139271
Edition 11

PHILIPS

INTENTIONALLY BLANK
INTENTIONNELLEMENT LAISSÉ EN BLANC
ABSICHTLICH LEER GELASSEN
OPZETTELIJK ONBEDRUKT GELATEN
ESTE ESPACIO SE HA DEJADO EN BLANCO DELIBERADAMENTE
PAGINA LASCIATA INTENZIONALMENTE VUOTA
PÁGINA DEIXADA EM BRANCO INTENCIONALMENTE
SIDAN AR AVSIKTILIGEN TOM
SKAL VÆRE TOM
JÄTETTY TARKOITUKSELLA TYHJÄKSI
DENNE SIDE ER TILSIGTET TOM
TĘ STRONĘ UMYŚLNIE POZOSTAWIONO PUSTĄ
ZÁMĚRNĚ PONECHÁNO PRÁZDNÉ
СТРАНИЦА ОСТАВЛЕНА ПУСТОЙ НАМЕРЕННО
Η ΣΕΛΙΔΑ ΑΥΤΗ ΠΑΡΕΜΕΙΝΕ ΣΚΟΠΙΜΩΣ ΚΕΝΗ
BOŞ BIRAKILMIŞTIR
刻意留白
刻意留空
このページは空白です
이 페이지는 공백 페이지입니다
หน้านี้ไม่มีข้อมูล
فارغ عن قصد

989803|3927|

HeartStart Training Pads II	
Instructions for Use (ENGLISH)	1
Electrodes de formation HeartStart II	
Mode d'emploi (FRANÇAIS)	11
HeartStart Schulungselektroden II	
Bedienungsanleitung (DEUTSCH)	23
HeartStart Trainingselektrode II	
Aanwijzingen voor gebruik (NEDERLANDS)	35
Electrodos de entrenamiento HeartStart II	
Modo de empleo (ESPAÑOL)	47
Elettrodi per addestramento HeartStart II	
Istruzioni per l'uso (ITALIANO)	59
Eletrodos de treinamento HeartStart II	
Instruções de Uso (PORTUGUÊS)	71
HeartStart-övningselektroder II	
Bruksanvisning (SVENSKA)	83
HeartStart Øvelseselektroder II	
Bruksanvisning (NORSK)	93
HeartStart-harjoitteluelektrodit II	
Käyttöohjeet (SUOMI)	103
HeartStart-træningselektroder	
Brugerhåndbog (DANSK)	113
Elektrody szkoleniowe HeartStart II	
Instrukcja obsługi (POLSKI)	125
Učební nalepovací elektrody HeartStart II	
Návod na použití (ČESKY)	137
Учебные электроды HeartStart II	
Руководство по эксплуатации (РУССКИЙ)	147
Εκπαιδευτικά pads II HeartStart	
Οδηγίες χρήσης (ΕΛΛΗΝΙΚΑ)	159
HeartStart Eğitim Pedleri II	
Kullanım Talimatları (TÜRKÇE)	171
HeartStart 訓練用電極片 II	
使用說明 (繁體中文)	181
HeartStart 培训用电极 II	
使用说明 (简体中文)	191
ハートスタート・トレーニング・パッド II	
取扱説明書 (日本語)	199
HEARTSTART 실습용 패드 II	
사용 설명서 (한국어)	211
แผ่นแปดสำหรับการฝึกหัด ฮาร์ทสตาร์ท II	
คำแนะนำการใช้งาน (ภาษาไทย)	221
231	
	لاصفات التدريب HeartStart II
	تعليمات الاستخدام (اللغة العربية)

ENGLISH: About This Edition

The information in this guide applies to the HeartStart Training Pads II 989803139271. This information is subject to change. Please contact Philips at www.philips.com/productdocs or your local distributor for information on revisions.

Edition History

Edition II

Publication date: August 2011

Publication number: 012654-001 I

Printed in Taiwan.

Copyright

© 2011 Philips Electronics North America Corp.

No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system or translated into any human or computer language in any form by any means without the consent of the copyright holder.

Unauthorized copying of this publication may not only infringe copyright but also reduce the ability of Philips Medical Systems to provide accurate and up-to-date information to users and operators alike.

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989803139271 HEARTSTART TRAINING PADS II

DESCRIPTION AND USE

The HeartStart Training Pads II are designed for use with the HeartStart FRx Philips 861304 and Laerdal 861305 automated external defibrillators. **These instructions are specific to use of the Training Pads II with an FRx Defibrillator.** Use of the Training Pads II with a HeartStart Trainer is described in the *Instructions for Use* provided with the Trainer.

The Training Pads II are reusable optional accessories for demonstration and occasional training applications. Replacement pads for the Training Pads II case are also available. **These pads cannot be used to deliver defibrillation treatment.**

Each training pads case contains a pair of reusable adhesive pads with integrated cable and connector. The case is packaged with:

- A fold-out Adult Pads Placement Guide. Metal areas on the guide allow the defibrillator to sense when the pads are properly applied.
- An Illustrated guide for using the pads, using the Pads Placement Guide, and repackaging the pads for future use.
- These *Instructions for Use*.

The HeartStart Training Pads II case and connector plug are yellow, to help prevent confusion with the gray case and blue connector of the SMART Pads II used for patient treatment. In addition, the training pads case has a red label that identifies its purpose as training.

Using the Training Pads II connected to a HeartStart FRx defibrillator, the user can choose any of eight emergency scenarios. The scenarios are described in the table on page 3. The scenarios are designed to let the user become familiar with how to operate the FRx defibrillator.

ADULT VICTIM. To demonstrate or practice using the FRx defibrillator to treat an adult victim of sudden cardiac arrest, use the HeartStart Training Pads II and the Adult Pads Placement Guide or a training manikin.

INFANT/CHILD VICTIM. To demonstrate or practice using the FRx defibrillator on an infant or a child under 55 pounds (25 kg) or 8 years old, use the HeartStart Training Pads II with an HeartStart Infant/Child Key [REF: 989803139311] and either an Infant/Child Pads Placement Guide [REF: 989803139281], or a training manikin. The Infant/Child Key and Pads Placement Guide are available separately from Philips.

When the training pads are connected and the Infant/Child Key is installed, the FRx defibrillator automatically goes into infant/child training mode. Its voice instructions – including coaching in performing cardiopulmonary resuscitation (CPR) – will be appropriate for a child victim.

NOTE: Before using the HeartStart FRx Defibrillator, the user should read all instructions provided with the defibrillator, including information about when the defibrillator should and should not be used. It is recommended that anyone who may use the HeartStart Defibrillator receive CPR/defibrillator training or other physician-authorized training.

NOTE: The HeartStart Training Pads II cannot deliver energy and therefore cannot be used for patient treatment. If the FRx defibrillator is needed for patient treatment during a training session, replace the training pads with a set of HeartStart SMART Pads II. Do not store the training pads with the defibrillator.

NOTE: Using the training pads require power from the FRx defibrillator battery. The battery supports 10 hours of use in the training mode, after which the battery must be replaced.

USING THE TRAINING PADS SCENARIOS

There are eight training scenarios available when using the training pads. They are described in the following table.

No.	Scenario Overview	Scenario Details
1	Shockable rhythm, one shock needed for conversion*	<ul style="list-style-type: none"> defibrillator detects a shockable rhythm, tells user to deliver a shock defibrillator detects a non-shockable rhythm**
2	Shockable rhythm, multiple shocks needed for conversion*	<ul style="list-style-type: none"> defibrillator detects a shockable rhythm, tells user to deliver a shock (repeated until an entire shock series is delivered) defibrillator provides pause for CPR defibrillator detects a shockable rhythm, tells user to deliver a shock defibrillator detects a non-shockable rhythm
3	Troubleshooting pads, one shock needed for conversion*	<ul style="list-style-type: none"> defibrillator detects poor pad contact, repeats pads placement instructions after one pad is removed and reapplied, defibrillator detects a shockable rhythm, tells user to deliver a shock defibrillator detects a non-shockable rhythm
4	Shockable rhythm, conversion, return to shockable rhythm, conversion*	<ul style="list-style-type: none"> defibrillator detects a shockable rhythm, tells user to deliver a shock defibrillator detects a non-shockable rhythm defibrillator detects refrillation (return to a shockable rhythm), tells user to deliver a shock defibrillator detects a non-shockable rhythm
5	Non-shockable rhythm	<ul style="list-style-type: none"> defibrillator detects a non-shockable rhythm throughout
6	Shockable rhythm, two shocks needed for conversion*	<ul style="list-style-type: none"> defibrillator detects a shockable rhythm, tells user to deliver a shock defibrillator still detects a shockable rhythm, tells user to deliver another shock defibrillator detects a non-shockable rhythm
7	Shockable rhythm, two shocks needed for conversion, return to shockable rhythm, conversion*	<ul style="list-style-type: none"> defibrillator detects a shockable rhythm, tells user to deliver a shock defibrillator still detects a shockable rhythm, tells user to deliver another shock defibrillator detects a non-shockable rhythm defibrillator detects refrillation (return to a shockable rhythm), tells user to deliver a shock defibrillator detects a non-shockable rhythm
8	Troubleshooting pads, two shocks needed for conversion*	<ul style="list-style-type: none"> defibrillator detects poor pad contact, repeats pads placement instructions after one pad is removed and reapplied, defibrillator detects a shockable rhythm, tells user to deliver a shock defibrillator still detects a shockable rhythm, tells user to deliver another shock defibrillator detects a non-shockable rhythm

* Conversion from ventricular fibrillation – the most common cause of sudden cardiac arrest – to a non-shockable rhythm.

** When the defibrillator detects a non-shockable rhythm, it instructs the user to care for the patient and provide CPR if needed.

Not all scenarios are appropriate for all user training. Training scenario number I is the default scenario and is most commonly used when teaching how to use the defibrillator to treat a victim of sudden cardiac arrest.

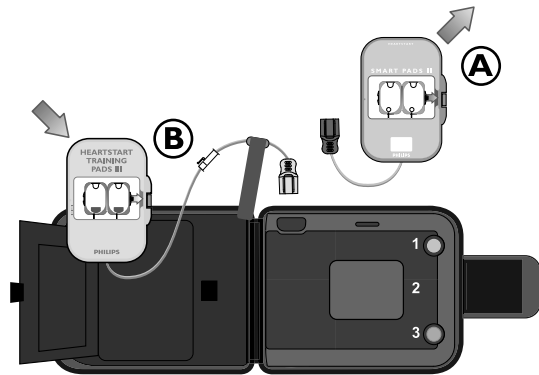
If the training scenarios are used as part of a formal training course, the instructor may wish to:

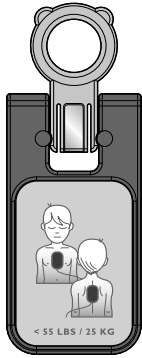
- select only the scenarios appropriate for the training goals,
- create case histories and patient details for the selected scenarios,
- highlight certain features of the defibrillator's performance,
- assure that users conform to local protocols,
- check pad placement, and
- evaluate user knowledge on topics such as CPR and assessing the patient.

USING THE HEARTSTART TRAINING PADS II TO CONNECT THE TRAINING PADS


To use the training pads, remove the case from any packaging in which it is provided or stored, then follow the steps below.

1. Be sure that the FRx defibrillator has a battery installed and a set of SMART Pads II connected and that the green Ready light is blinking.
2. Unplug the SMART Pads II connector from the pads connector port on the front of the FRx Defibrillator (A). Do not open the pads case. Set it aside for reconnection after completion of training.
3. Plug the connector of the Training Pads II into the connector port on the front of the FRx. Place the pads case in the pocket provided inside the carry case (B).

Infant/Child Key

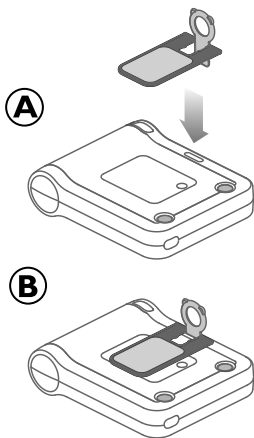As soon as the training pads are connected, the FRx announces “TRAINING.” The blue i-button on the defibrillator starts flashing. (If the Infant/Child Key is inserted, the defibrillator will say “INFANT/CHILD MODE” when the On/Off button is pressed.)

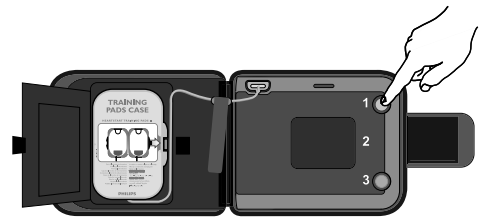
As a safety precaution, the defibrillator will instruct you to remove the training pads and connect a set of SMART Pads II in case of an emergency. It will then invite you to press the flashing blue button to choose a training scenario.

NOTE: If you do not press the blue i-button within 30 seconds after connecting the training pads, the defibrillator will go to standby training mode and the green Ready light will blink.

To train in infant/child defibrillation, insert the Infant/Child Key into the slot at the top center of the front panel of the FRx. The pink portion of the Infant/Child Key pivots (A) and fits into the slot (B), with the front of the key lying flat on the surface of the FRx so the pediatric pads placement diagram is visible. The Infant/Child Key can be inserted either before or after the training pads are connected to the FRx. If the Key is inserted after a scenario is running, the scenario will restart from the beginning.

TO SELECT A TRAINING SCENARIO

The default training scenario is number 1. After plugging in the training pads, press the On/Off button to use this scenario. The defibrillator will not announce the scenario number.


To select a different scenario, after connecting the training pads, press the flashing blue i-button to scroll through the eight scenarios in sequence. The defibrillator announces each scenario by number as you press the button. When you reach the scenario number you want, press and hold the On/Off button until the defibrillator turns off, or wait for 10 seconds and it will automatically go into standby training mode.

To select a training scenario when the device is already in training standby mode (the training pads are connected and the green Ready light is flashing), press the blue i-button to access scenario selection, then press it again to scroll through the scenarios and make your selection.

NOTE: As long as the battery and training pads are not removed, you can delay the training session using the selected scenario for up to one hour. After one hour, the blue i-button will start blinking, and the defibrillator will start chirping to alert the user to either continue the training use or unplug the training pads and connect a set of HeartStart SMART Pads II.

TO RUN A TRAINING SCENARIO

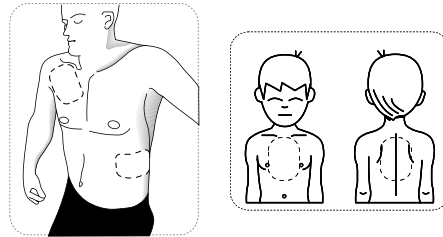To run the selected training scenario, treat the training pads exactly as you would the SMART Pads II, using the Adult or Infant/Child Pads Placement Guide (shown at right) or a CPR training manikin with an External Manikin Adapter M5089A to


simulate the patient. (Illustrated instructions for applying the manikin adapter to a training manikin are provided with the Adapter.)

1. If you are training in infant/child defibrillation, insert the Infant/Child Key into the defibrillator.
2. Press the On/Off button to turn on the defibrillator.
3. Remove the HeartStart Training Pads II case from the carry case. If using a clothed manikin, remove the clothing as instructed.
4. Open the pads case and remove one pad.

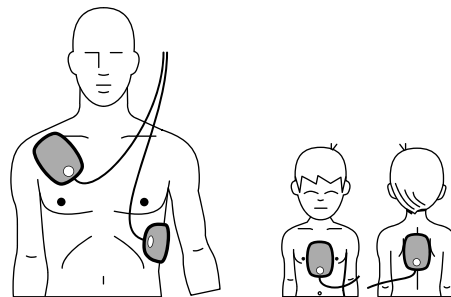5. Place the pad on the pads placement guide or the manikin, exactly as shown. The pads icons on the FRx front panel – or, if the Infant/Child Key is inserted, on the front of the Key – will be flashing, to help guide you. (Note that pad

placement for infants and young children differs from placement on adults.) Press the pad down firmly.

6. Then remove the other pad from the case and place it. (It does not matter which pad is placed first or which pad is placed in which position.)
*PRECAUTION: Do not place the pads on a person.**
7. Continue to follow the FRx instructions for remainder of scenario.
8. When the scenario has been completed, or to stop a scenario in process, press the On/Off button until the defibrillator turns off.
9. To repeat the same scenario, replace the pads in their case, then begin again by pressing the On/Off button. As long as the battery has not been removed, the defibrillator will automatically use the previously selected scenario. To select a different scenario, press the i-button. Each time the battery is inserted, the defibrillator will default to scenario number 1.

NOTE: To turn off the defibrillator at any time, press and hold the On/Off button until the defibrillator turns off.

DEFIBRILLATOR PERFORMANCE DURING TRAINING

When a training scenario is running, the defibrillator behaves exactly as it would during actual emergency use, providing voice instructions, one or more simulated shocks, pauses for CPR, optional CPR guidance, etc.†

For safety reasons, certain defibrillator features are not functional while the training pads are installed:

- The defibrillator cannot be used for actual patient care; it cannot analyze patient heart rhythm or deliver a shock.
- It will not perform the battery insertion and periodic self-tests.
- If it is not used (no buttons are pressed and the connector is not removed) for one hour with the training pads connected, the defibrillator will start “chirping” to alert the user to either continue the training use or remove the training pads and connect a set of HeartStart SMART Pads II.

* A Precaution is a condition, hazard, or unsafe practice that could result in minor personal injury.

† If the FRx configuration has been changed from the default settings, the modifications are applied to its behavior during the training simulations.

- If the i-button is pressed while the defibrillator is in training standby mode, the defibrillator will announce that it is training. If the i-button is pressed a second time, the defibrillator will tell you the number of the current training scenario. A third press of the i-button will allow you to select another scenario.

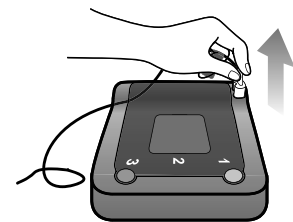
NOTE: If the battery power becomes low during training, the defibrillator will tell you to replace the battery immediately. It will begin chirping, and the blue i-button will start blinking. If you press the blue i-button or the green On/Off button, the defibrillator will repeat the instruction to replace the battery.

AFTER USING THE HEARTSTART TRAINING PADS II


After you have finished using the FRx for training, press and hold the On/Off button to turn off the defibrillator. Remember that using the training pads requires battery power. The defibrillator battery is designed to support 10 hours of training use. If battery power is low after training, the defibrillator will chirp to alert you. Be sure to replace the battery with a new M5070A battery.

Prepare the training pads for their next use as follows:

1. Unplug the training pads connector from the defibrillator and plug in a set of SMART Pads II to be sure the defibrillator is ready for use in an emergency.
2. Remove each pad from the Pads Placement Guide or manikin and replace it inside the HeartStart Training Pads II case (A). Press the pads firmly into the case (B).

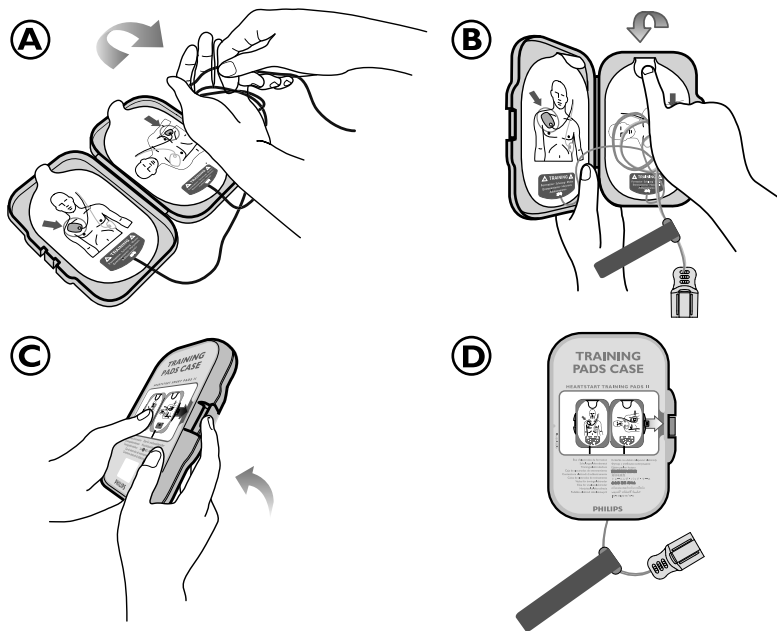3. Loosely wind the pads cable and place it inside the case (A). Fold down the tab at the top of each pad (B), and close the case (C), allowing the connector plug end and a few inches of cable to hang outside the case (D).

NOTE: To avoid confusion when using the defibrillator in an emergency, be sure to store training pads case separately from the defibrillator.

MAXIMIZING SERVICE LIFE

The HeartStart Training Pads II are designed for up to 100 uses. To help ensure continued performance, be sure to return the pads to their case after each use. Avoid leaving the pads case in heat or direct sunlight.

When the original training pads have reached the end of their service life, you can order 989803139291 Replacement HeartStart Training Pads for use with your original training pads case.

NOTE: Please contact your local HeartStart distributor if you have any questions about using the training pads or need technical support. In the United States, please call 1-800-263-3342.

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PHILIPS

Philips Medical Systems is part of Royal Philips Electronics

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Philips Medical Systems ingår i Royal Philips Electronics

Philips Medical Systems er en del av Royal Philips Electronics

Philips Medical Systems on osa Royal Philips Electronics - konsernia

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Philips Medical Systems, Royal Philips Electronics'in bir koludur

Philips Medical Systems 屬於 Royal Philips Electronics 旗下

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Philips Medical Systems

United States

Philips Medical Systems
2301 Fifth Avenue, Suite 200
Seattle, WA, 98121-1825, USA
(800) 263-3342

Canada

Philips Healthcare, a Division of Philips
Electronics Ltd.
281 Hillmount Road
Markham, Ontario
L6C 2S3, Canada
(800) 291-6743

Europe, Middle East, and Africa

Philips Medizin Systeme Boeblingen GmbH
Cardiac and Monitoring Systems
Hewlett-Packard Strasse 2
71034 Boeblingen, Germany
(+49) 7031-463-2254

Latin America

Philips Medical Systems Ltda.
Rua Verbo Divino - 1400 - 7o Andar
04719-002 -Sao Paulo, Brazil
0800 7017789

Asia Pacific

Philips Electronics Hong Kong Ltd.
6/F, Core Building I
1 Science Park East Avenue
Hong Kong Science Park
Shatin, New Territories, Hong Kong
(852) 2821 5888

REF 989803139271

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Niets uit deze uitgave mag in enige vorm of op enige manier worden verveel- voudigd, verzonden, getranscribeerd, opgeslagen in een geautomatiseerd gegevensbestand of vertaald in menselijke of computertaal zonder toestemming van de houder van het copyright.